

GB

CEMEL "U" FORM®

Articulating Paper

Directions For Use

Indications;

Marking the occlusal condition of natural teeth, artificial teeth and other restoratives.

Direction for Use;

1. Dry out occlusal surface of tooth structure (artificial teeth and other restoratives).
2. To adjust occlusal contact by grinding on higher contact area, bite several times at centric occlusion and grind color marked area.
3. To diagnosis difference of occlusal contact on each mandibular position parallel, occlusal contact for each mandibular position may be easily recognized with marking by different colored product.


- Double sided, 80 µm thickness.

- Easy, quick and precise marking

- No need to twizzers.

- Two dark colors (Blue/red) are available for good marking.

Technical Specifications

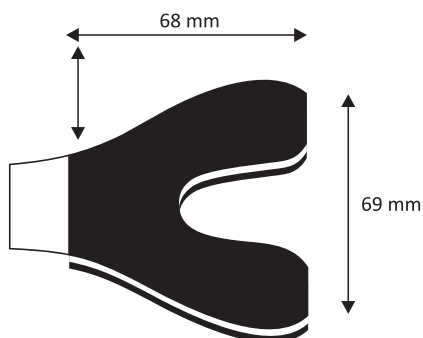

Package Details

6 Booklets of 12 Sheets



PRESIDENT DENTAL GmbH
Kesselbodenstrasse 5, 85391 Allershausen, Germany
Tel: +49 8166 389 9 820 • Fax: +49 8166 389 9 821
www.presidentdental.com • info@presidentdental.com

GB**CEMEL "I" FORM[®]**

Articulating Paper

Directions For Use**Indications;**

Marking the occlusal condition of natural teeth, artificial teeth and other restoratives.

Direction for Use;


1. Dry out occlusal surface of tooth structure (artificial teeth and other restoratives).
2. To adjust occlusal contact by grinding on higher contact area, bite several times at centric occlusion and grind color marked area.
3. To diagnosis difference of occlusal contact on each mandibular position parallel, occlusal contact for each mandibular position may be easily recognized with marking by different colored product.

- Double sided, 80 µm thickness.

- Easy, quick and precise marking

- No need to twizzers.

- Two dark colors (Blue/red) are available for good marking.

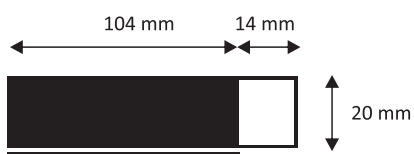Technical Specifications**Package Details**

6 Booklets of 12 Sheets



PRESIDENT DENTAL GmbH
Kesselbodenstrasse 5, 85391 Allershausen, Germany
Tel: +49 8166 389 9 820 • Fax: +49 8166 389 9 821
www.presidentdental.com • info@presidentdental.com